



Gentaur Europe BVBA  
Voortstraat 49,  
1910 Kampenhout  
BELGIUM

Tel 0032 16 58 90 45  
[info@gentaur.com](mailto:info@gentaur.com)  
<https://maxanim.com/sd-bioline/>

# SD BIOLINE H.pylori Ag

## One Step *H.pylori* Antigen Test

*Helicobacter pylori* (*H. pylori*) is a spiral-shaped bacterium and that is found in the gastric mucous layer or adherent to the epithelial lining of the stomach. *H. pylori* causes more than 90% of duodenal ulcers and up to 80% of gastric ulcers. Approximately two-thirds of the world's population is infected with *H.pylori*. SD BIOLINE H.pylori Ag test thanks to its accuracy and the ease of use aids fast and accurate diagnosis of *H. pylori* infection.



### General Information

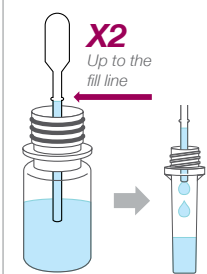
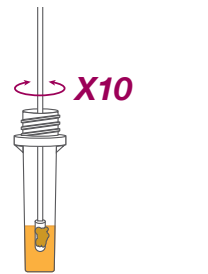
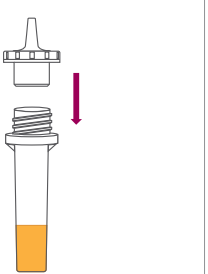
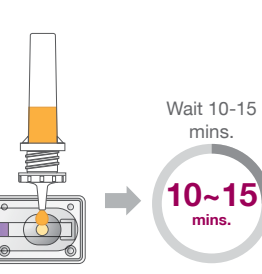
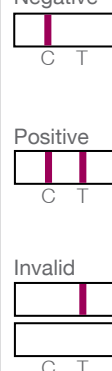
SD BIOLINE H.pylori Ag kit is a rapid, qualitative test for the detection of *Helicobacter pylori* antigen in human fecal specimens.

- Easy to use: No need of any equipment
- Test result: 10 -15 minutes
- Shelf life and storage temperature: 24 months at 1-30 °C
- Performance: Sensitivity-98.4 %, Specificity-100 %
- **Specimen: Fecal specimens**

### Materials Provided

- Test device
- Sample collection tube
- Assay diluent
- Sample collection swab
- Disposable dropper
- Disposable dropping cap

### Test Procedure

- 1 Add Assay Diluent**  
Transfer assay diluent twice.  

- 2 Insert swab sample**  
Insert the swab into the sample collection tube and swirl the swab at least 10 times.  

- 3 Assemble dropping cap**  
Assemble dropping cap on the sample collection tube.  

- 4 Add Specimen**  
Dispense 3 drops.  

- 5 Read Results**  

  - Negative
  - Positive
  - Invalid

### Ordering Information

Product	Cat. No.	Type	Specimen	Pack Size
H.pylori Ag	04FK20	Device	Fecal	20T/Kit